IT

Comprese detergenti

Contenuto

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Cleaning Tablets

Content

28 Antimicrobial Cleaning Tablets, strips with 7 tablets each (CE 0044)
1 Instruction leaflet

Field of application

For the external care and disinfection of earmoulds, to maintain performance and prolong the life of BTE earmoulds effectively.

Rate of use

We recommend daily cleaning. Only a daily cleaning prolongs the life of your high-quality hearing system and guarantees proper hygiene and functioning.



Effect

Antimicrobial formula: The cleaning tablets, in the practical 7-tablet strip, provide protection from bacteria and fungus reducing the risk of infection and inflammation of the ear, day after day. A constituent of the formula protects children from accidental ingestion by making them taste bitter. The bitter constituent does not affect the cleaning performance.

Composition

OtoVita® Cleaning Tablets contain: Potassium monopersulphate. Advanced protection from bacteria and fungus, tested with Staphylococcus aureus, Pseudomonas aeruginosa, Streptococcus mutans and Candida albicans. Contact time: 30 minutes after dissolving of the tablet.

Storage at room temperature 18 °C-28 °C (64 °F-82 °F).
Date of information: January 2010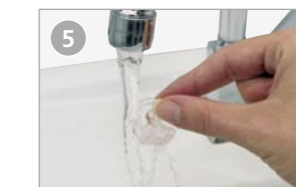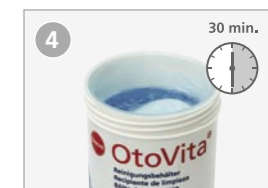

050510-27000-3206-1